


# Did you know? Glioblastoma

## Abstract

Glioblastoma accounts for 15 to 20% of brain tumors. Its average survival is 18 months. 70% of glioblastomas occur between 45 and 70 years old. Glioblastomas predominate in men in a ratio of 1.5 to 1.8 / 1 (Ref.1).

Glioblastoma is the most common brain tumor in adults (Figure 1) with an annual incidence of 3.0 to 3.6 cases per 100,000, corresponding to 240,000 new cases per year worldwide. Ranked grade IV glioma by the World Health Organization (WHO), its prognosis is poor, with a 5-year survival rate of 3%.

Nearly 90% of glioblastomas are primary tumors (de novo), characterized by a short clinical history (<3 months) with no precedent of glioma; the remaining 10% are secondary to the anaplastic transformation of an infiltrating Grade II or III glioma.

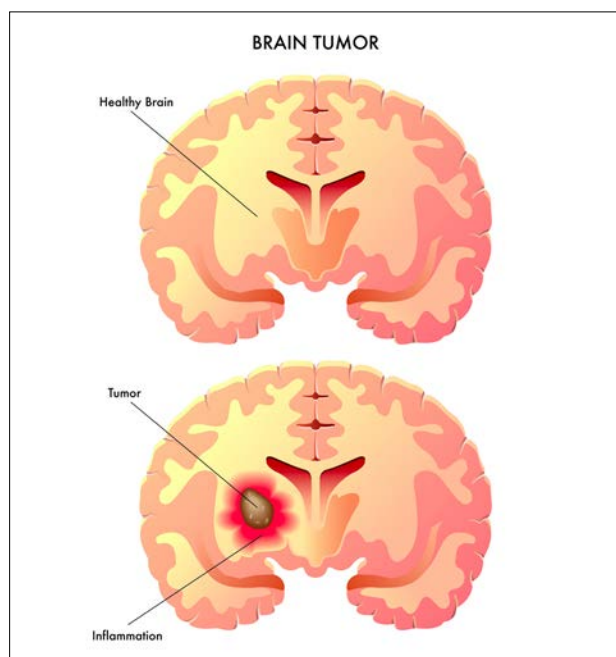

**Figure 1**

Histologically, we find a neo-angiogenesis, known to be an important element of aggressiveness of these tumors, reflecting their degree of malignancy. These new vessels are abnormal, fragile and permeable, contributing to peri-tumoral edema and increased risk of bleeding.

Imaging plays a central role in the diagnosis and follow-up of glioblastomas. In 1990, Macdonald introduced the first classification, revised and now improved by the RANO criteria ("Revised Assessment in Neuro-Oncology") in 2010.

## Complications

- Hemorrhage
- Hydrocephalus
- Cerebral commitment
  - Temporal commitment
  - Extension to cerebellar tonsils in the occipital fossa
- Tumor meningitis
- Thromboembolic pathology
- Infectious complications

## Possible treatments

- Surgery, including a more or less complete excision of the tumor
- In situ chemotherapy with Carmustine implants (Gliadel®)
- Radiotherapy 4 to 6 weeks after surgery
- Association of Nitrosourea chemotherapy such as Temozolomide (Temodal®)
- Second-line chemotherapy with VEGF inhibitors, Bevacizumab (Avastin®)
- New type of immunotherapy-based treatment

## In Olea Sphere®

The Perfusion (DSC) and Diffusion plugins can be used in addition to conventional sequences to assess brain tumors.

In this Glioblastoma case (Figure 2) we note an increase of rCBV to 7.57 mL/100mL at the level of the fleshy tissue of the tumor, 8 times higher than the healthy contralateral value. The presence of necrosis being noticeable, a T1 perfusion (DCE) was also added to the protocol, increased AUC (Area Under the Curve) and Ktrans values can be found.

A texture analysis with the Entropy study makes it possible to assess the spectrum of singularities of the image of the different levels of gray (Figure 2).

The longitudinal analysis plugin makes it possible to compare all sequences and parametric maps over time using a suitable registration. We can follow, in this case, the evolution of rCBV, already increasing on the first examination in March, without any contrast enhancement on T1 post gadolinium (Figure 3).

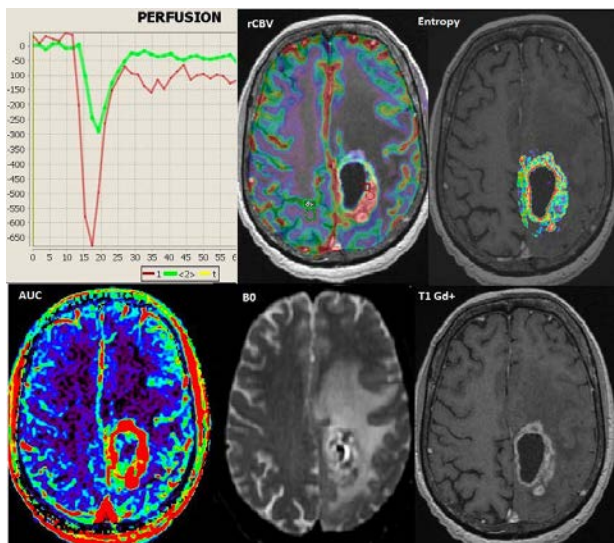


Figure 2

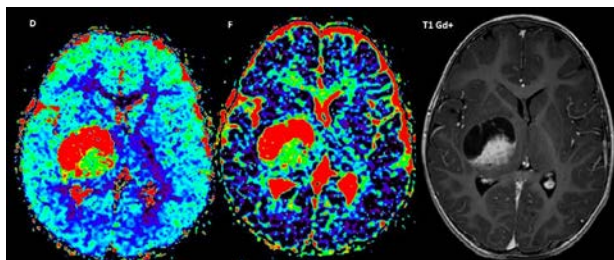


Figure 4

## Addition

rCBV "relative Cerebral Blood Volume" is a good marker of tumor grade and useful during patient monitoring to discriminate recurrence from pseudo progression.

The volumetric study of tumors seems to be robust and promising, compared to two-dimensional measurements recommended today in the follow-up (Figure 3). IVIM (intravoxel incoherent motion) parameters, calculated using diffusion imaging, are new useful biomarkers in the analysis of brain tumors (Figure 4).

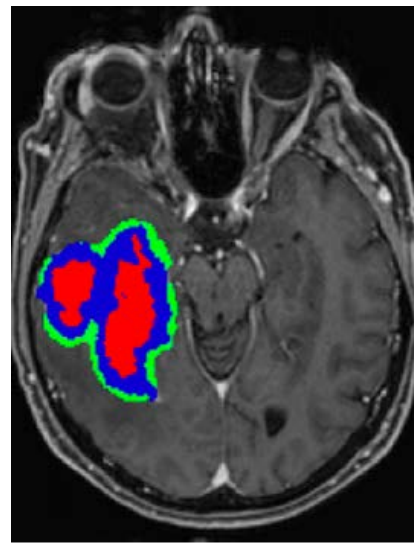


Figure 3

## References

1. Baldi I, Huchet A, Bauchet L, Loiseau H. [Epidemiology of glioblastoma]. *Neurochirurgie* 2010;56:433-40
2. <https://link.springer.com/article/10.1007%2Fs00401-0050991-y>
3. <https://www.ncbi.nlm.nih.gov/pmc/articles/PMC4248049/>
4. <https://www.ncbi.nlm.nih.gov/pmc/articles/PMC3095028/>
5. <https://www.ncbi.nlm.nih.gov/pubmed/27461038>
6. <http://ascopubs.org/doi/full/10.1200/JCO.2017.72.7511>
7. <https://www.sciencedirect.com/science/article/pii/S1064968916300459?via%3Dihub>
8. [http://neurochirurgica.org/IMG/article\\_PDF/Glioblastomes\\_a22.pdf](http://neurochirurgica.org/IMG/article_PDF/Glioblastomes_a22.pdf)
9. <http://www.ipubli.inserm.fr/bitstream/handle/10608/4820/?sequence=77>
10. <https://www.ncbi.nlm.nih.gov/pmc/articles/PMC4936699/>
11. [http://www.medecine.ups-tlse.fr/dcem3/module15/neurologie/146\\_poly\\_tumeurs\\_intracraniennes.pdf](http://www.medecine.ups-tlse.fr/dcem3/module15/neurologie/146_poly_tumeurs_intracraniennes.pdf)
12. [http://sfncongres.net/pdf/presentations2014/jeudi\\_03-04\\_auditorium/14H55\\_Savatovsky.pdf](http://sfncongres.net/pdf/presentations2014/jeudi_03-04_auditorium/14H55_Savatovsky.pdf)

**OLEA MEDICAL®**  
[www.olea-medical.com](http://www.olea-medical.com)

Olea Sphere® v3.0, medical imaging post-processing software, is a medical device manufactured and marketed by Olea Medical®. This medical device is reserved for health professionals. The software has been designed and manufactured according to the EN ISO 13485 quality management system. Read the instructions in the notice carefully before any use.

Instructions for Use are available on <http://www.olea-medical.com/en/> Manufacturer: Olea Medical® S.A.S. (France). Medical devices Class IIa / Notified body: CE 0459 LNE-GMED.

**IMPROVED  
DIAGNOSIS  
FOR LIFE**